

embrace[®]
aspectimaging



embrace[®]
Neonatal MRI System

Transforming Neonatal Imaging
Inside the NICU

Parent peace of mind

Moving a baby out of the environmentally sensitive NICU is stressful for parents. With the Embrace®:

- Babies remain in the safe and familiar NICU environment
- Parents can stay with baby during the scan
- Video display system allows continuous visual contact throughout scan
- No more worrying about associated risks of off-unit scanning



A perfect fit

Compared to traditional MRIs, the Embrace® has a smaller footprint designed to fit inside the NICU.

Additional features include:

- Small footprint designed to be easily placed in the NICU
- Unique self-shielded magnet can be placed near NICU equipment and does not require a zone 4 safety room
- Non-cryogen technology does not require a cooling system



It's quiet

Managing noise levels can be challenging in the NICU environment. The Embrace® with Whisper Scan technology reduces sound levels in both the MRI suite and within the system during the neuro scan.

- Completely silent when not scanning
- 67 dB average ambient sound levels in the MRI suite during scanning, meeting the sound specifications for NICU
- Max patient exposure is 85 dB, significantly lower than traditional MR systems



aspect imaging

embrace[®]
aspect imaging

PHILIPS

PHILIPS



Embrace® some industry firsts

- The world's first FDA cleared, CE marked compact MRI system built for use inside the NICU
- First system designed to support specific needs of neonatal patients

A gentle design for your tiniest patients

The innovative Embrace® design delivers:

- Ready-to-scan technology at the point of care
- Whisper Scan sequences that reduce noise levels
- Minimized patient disruption and interruption of care
- Thermally controlled patient bed


Minimize scheduling roadblocks

Timing is critical when scheduling an MRI scan for a NICU baby. With Embrace® you can:

- Schedule scans within minutes of an order
- Scan when the baby is stable
- Improve radiology department throughput and productivity

Save time and simplify your workflow

On average, it can take several hours to prep, transport, scan and return a baby to the NICU. MRI scanning inside the NICU accelerates workflows up to seven times faster* compared to off-unit scanning. With the Embrace[®], the entire scanning process may be completed in under an hour.

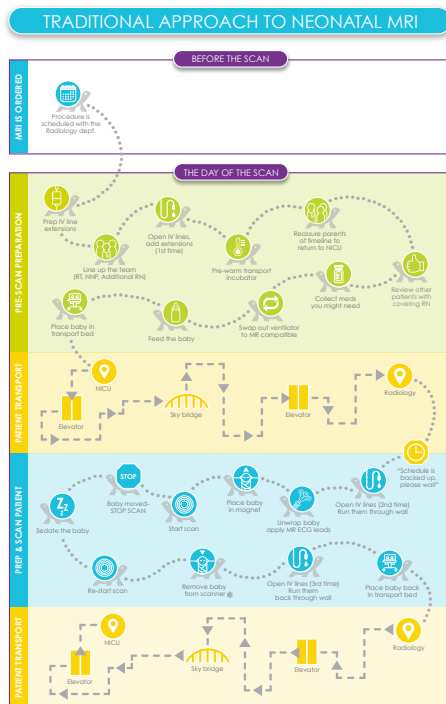

Which approach would you choose?
Which would your babies choose?

OR



Scan and see for yourself how the patient-focused Embrace[®] Neonatal MRI System enables safer, quicker and more efficient imaging right *inside* the NICU, compared to traditional off-unit MR scanning practices.

WATCH VIDEO



*Bin-Nun, A (2019, Feb). Global Brain Injury scores in Preterm Infants: Validation of a novel 1T Neonatal MRI vs Conventional 1.5T MRI. Podium Presentation at the 10th Annual Neonatal Brain Monitoring & Neuroprotection Conference, Tampa, FL.

Minimizing the risk and complexity of neonatal neuro imaging

When it comes to NICU babies, the only time they should be moved outside of the NICU is when they are discharged from the hospital. Moving vulnerable babies to off-unit radiology departments can increase infection risk exposure and patient safety issues associated with transport. Now more than ever, the need to keep babies safe inside the NICU, until discharge, is a necessity, and should be accomplished even when accessing MR imaging.

Prepping and transporting a baby for an MRI scan is time consuming, depletes staff resources and creates added stress for the baby, the parents, and the care team. Placing the Embrace[®], a dedicated MRI system exclusively designed for neonatal imaging inside the NICU, is a safer and more efficient solution that eliminates numerous risks associated with off-unit transport.

Keeping NICU babies in their comfort zone

Designed for the critical needs of high-risk neonates, the Embrace[®] is equipped with unique features not found on adult MRI scanners:

- Thermally controlled patient bed with closed-loop air circulation system to maintain baby's temperature
- Quieter noise levels compared to traditional scanners
- Tubing management system accommodates IV lines, respiratory circuits and monitoring leads


Keeping clinical teams in their comfort zone

The NICU is a highly customized environment designed, equipped and staffed for the special needs of critically ill babies. Moving MRI into the NICU helps to:


- Eliminate staff coverage requirements for off-unit transport
- Quickly access all necessary resources, when you need them, inside the NICU
- Keep specialty support teams, such as respiratory therapy, in the NICU to avoid understaffing issues

Embrace®: a patient-focused approach


| | | | |
|---|--|---|--|
| Magnet: | | Gradient System: | |
| <ul style="list-style-type: none"> • Fixed permanent magnet • Field strength 1.0T • Weight: 5,500 kg • Patient accessible bore size: 184 mm W x 260 mm H • Iron-based magnetic shielding • Passive and active shimming • 0 external magnetic field • 5 Gauss Line confined within system cover • H: 71 in (181 cm) • W: 57 in (145 cm) • L: 67 in (171 cm) | | <ul style="list-style-type: none"> • 150m T/m peak gradient strength • Slew rate 500 T/m/Sec • Fastest rise time 0.3 mSec | |
| | | Pulse Sequences: | |
| | | <ul style="list-style-type: none"> • 2D SE T1 • 2D FSE T2 • 2D ADC Map SPLICE (Diffusion) • 2D ADC Map SE (Diffusion) • 2D/3D GRE (T1) • 3D MPRAGE (T1) | |
| Imaging: | | Connectivity: | |
| <ul style="list-style-type: none"> • Field of view is an ellipsoid: 120 mm (horizontal) x 130 mm (vertical) x 130 mm (depth) • Minimum achievable slice thickness 2D: 1.5 mm • In-plane sampling resolution 2D, 3D: 16-512 px • Minimal imaging voxel size: 0.3 x 0.3 x 0.3 mm³ | | <ul style="list-style-type: none"> • PACS/HIS/RIS connectivity with DICOM compatibility • MR workstation supports Modality Work List and multiple PACS systems | |
| RF Head Coil: | | Patient Specifications: | |
| <ul style="list-style-type: none"> • Maximum head circumference: 38 cm • Transmit-receive head coil with integrated connector designed specifically for infants • Solenoid design for optimal signal-to-noise with the magnet's horizontal static magnetic field • RF coil inner diameter: 143 mm | | <ul style="list-style-type: none"> • Accommodates babies weighing 1 to 4.5 kg • Maximum head circumference: 38 cm • Designed for both intubated and non-intubated patients | |
| | | Acoustic Noise: | |
| | | <ul style="list-style-type: none"> • System acoustic output (in room): Average 69 dB(A), peak 71 dB(A) • Patient acoustic output (in magnet): Average 85 dB(A), peak 87 dB(A) | |

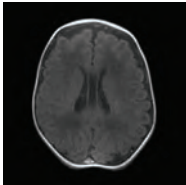


AX T1 SE

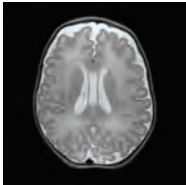


AX T2 FSE

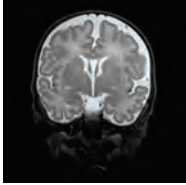


COR T2 FSE

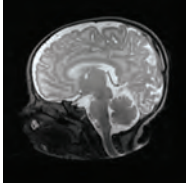


SAG T2 FSE

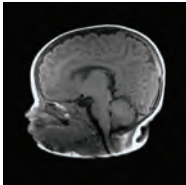

SAG T1 SE

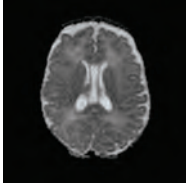

ADC Map

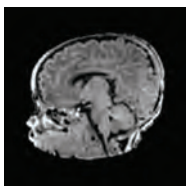

3D GRE

Become a part of the transformation.

Discover more at embracemri.com



Aspect Imaging, Ltd.
3200 West End Avenue
Suite 500
Nashville, TN 37203

Call: +1 866 609 1554
Email: info@embracemri.com
Web: www.embracemri.com

Embrace® is a registered trademark of Aspect Imaging, Ltd. Aspect Imaging is a global leader in the design and development of compact, high-quality MR imaging solutions, designed for use in pre-clinical research and medical applications.